

DIRECTIONS FOR USE

exofin[®]
HIGH VISCOSITY TISSUE ADHESIVE

1 The incision or trauma site must be clean and dry before applying adhesive. Ensure that the wound edges are easily approximated.

2 Screw the Applicator Tip onto the threaded tube in a clockwise direction until the applicator is tight and seated on the tube. The applicator will puncture the foil membrane on the tube once seated and adhesive will be allowed to flow.



2



3

3 While holding the tube upright and away from the patient, gently squeeze the tube from the bottom, moving the adhesive upward until it becomes visible through the applicator device.

4 While approximating skin edges, apply adhesive to the wound site by squeezing the aluminum tube gently and continuously in a back and forth motion while spreading over wound site.

5 Continue to hold the wound edges in approximation until the adhesive becomes tacky. Typically less than 20 seconds. A smooth and even coat of adhesive is desirable.



4



5

6 Once adhesive has completely polymerized (non-tacky) you may cover the site with a secondary bandage. If a secondary bandage is used, DO NOT apply to the adhesive area until it is completely tack free. Applying while tacky may result in adhesive removal when removing secondary bandage and possibly dehiscence of the wound site.

7 Discard adhesive device according to normal protocol after use.