



DIRECTIONS FOR USE

exofin[®] PRECISION
PEN

- 1** The incision or trauma site must be clean and dry before applying adhesive. Ensure that the wound edges are easily approximated.
- 2** Tighten the pre-assembled Applicator Tip onto the pen body in a clockwise direction until the applicator is tight. The applicator will puncture the foil membrane inside the pen once seated and adhesive will be allowed to flow.
- 3** While holding the pen slightly downward and away from patient, begin squeezing until adhesive is visible in the connector. Continue squeezing until glue appears in the clear applicator tip.
- 4** Immediately begin to apply **exofin**[®] PRECISION PEN Topical Skin Adhesive to the wound site. Continue squeezing while brushing in a back and forth motion “painting the surface” until completely covered. If additional sites require adhesive, move to those sites and repeat.
- 5** The adhesive should be tack free approximately 30 seconds after application, but is dependent upon the thickness of the adhesive layer. Care should be taken to create a smooth even layer of adhesive to prevent thick edges which may lead to inadvertent bond breakage of the adhesive.
- 6** Once adhesive has completely polymerized (non-tacky) you may cover the site with a secondary bandage. If a secondary bandage is used, **DO NOT** apply to the adhesive area until it is completely tack free. Applying while tacky may result in adhesive removal when removing secondary bandage and possibly dehiscence of the wound site.

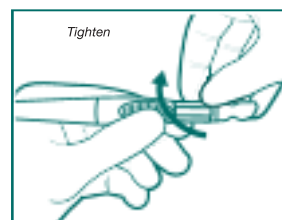


2

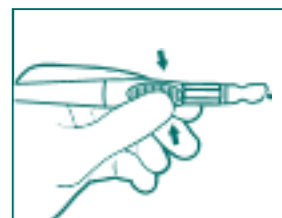

3



4

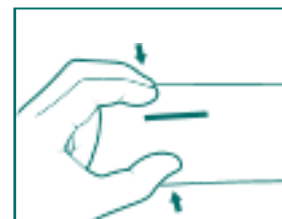

5