

TAW Trolley Stand

Model: TC201

User Manual

Content

1 Safety Information.....	2
1.1 Dangers	2
1.2 Warnings	2
1.3 Cautions.....	3
1.4 Note	3
2 Equipment Symbols	3
3 Application Areas / Intended Use	4
4 Reporting Obligation	4
5 Customer Service	4
7 Manual Safekeeping	5
8 Specification.....	5
9 TAW Trolley Stand Parts Schematic.....	6
10 Assemble the TAW Trolley Stand	7
11 Maintenance and Adjustment.....	8
12 Cleaning and Disinfection	8

1 Safety Information

Caution: *Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.*

1.1 Dangers

There is no hazard-level safety information in this manual.

1.2 Warnings

WARNING

- **Do not dismantle the wall frame by yourself.**

The maintenance of the TAW Trolley Stand must be carried out by maintenance personnel trained and authorized by the company.

- **Disposal of packaging materials**

The relevant local regulations or the hospital's waste disposal system must be observed. The packaging materials must be placed out of the reach of children.

- **Electric shock hazard**

After installation, the electric cables of MD and MD systems may only be connected to the power source and the medical device. Before maintenance or disassembly all power cables must be disconnected from the power source.

1.3 Cautions



- **Installation of medical electrical equipment**

The medical device may only be installed if all screws are tightened and securely fastened.

1.4 Note

Notice

- **Make sure the SourceMark VLMA Display is properly hung before pushing the TAW Trolley Stand.**

2 Equipment Symbols

	Attention: Consult accompanying documents (this manual).		Serial number
	Read Operator's Manual		Importer
	Manufacturer		Medical Device
	Manufacture date		U.S. Agent

Rx only	Prescription Use Only		
----------------	-----------------------	--	--

3 Application Areas / Intended Use

TAW Trolley Stand are intended to install and position medical electrical equipment (medical device) and its accessories onto this equipment and used as auxiliary equipment for doctors and medical staff in operating rooms and treatment rooms.

4 Reporting Obligation

Any serious incidents associated with the device should be reported to the manufacturer and the competent authorities in the Member State where the user and/or patient is located.

5 Customer Service



Zhejiang UE Medical Corp.

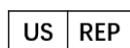
No.8, Youyi Road, Baita Economic Development Zone, Xianju, Zhejiang 317300, China

Tel: +86-576-87788798

Fax: +86-576-87721129

Website: www.ueworld.com

E-mail: export@ueworld.com



Importer :

SourceMark, LLC

Franklin, TN 37069 USA.

7 Manual Safekeeping

For any questions that may arise at a later time please safe-keep our manual!

8 Specification


8.1 Hardware Specifications

Model	TC201
Brand	SourceMark
Load Capacity	10kg
Angle of pitch	270°
Adjust Way	Mechanical lifting
Basket	300*200*150mm, Plastic, White color as standard
Castor	3inch, All with brakes
Material	Aluminum alloy+Sheet metal+ABS
Package Size	<u>87*50*25cm</u>
Certificates	ISO13485/FDA





8.2 Environmental Specifications

Temperature range	Operating	0 °C ~+40 °C
	Storage	-40°C~+70°C
Humidity range	Operating	15% - 95 %, non-condensing
	Storage	10% - 95 %, non-condensing
Altitude range	Operating	-500 ~ +4,600 m (57.0kPa~106.0kPa)
	Storage	-500 ~ + 4,600 m (50.0kPa~106.0kPa)

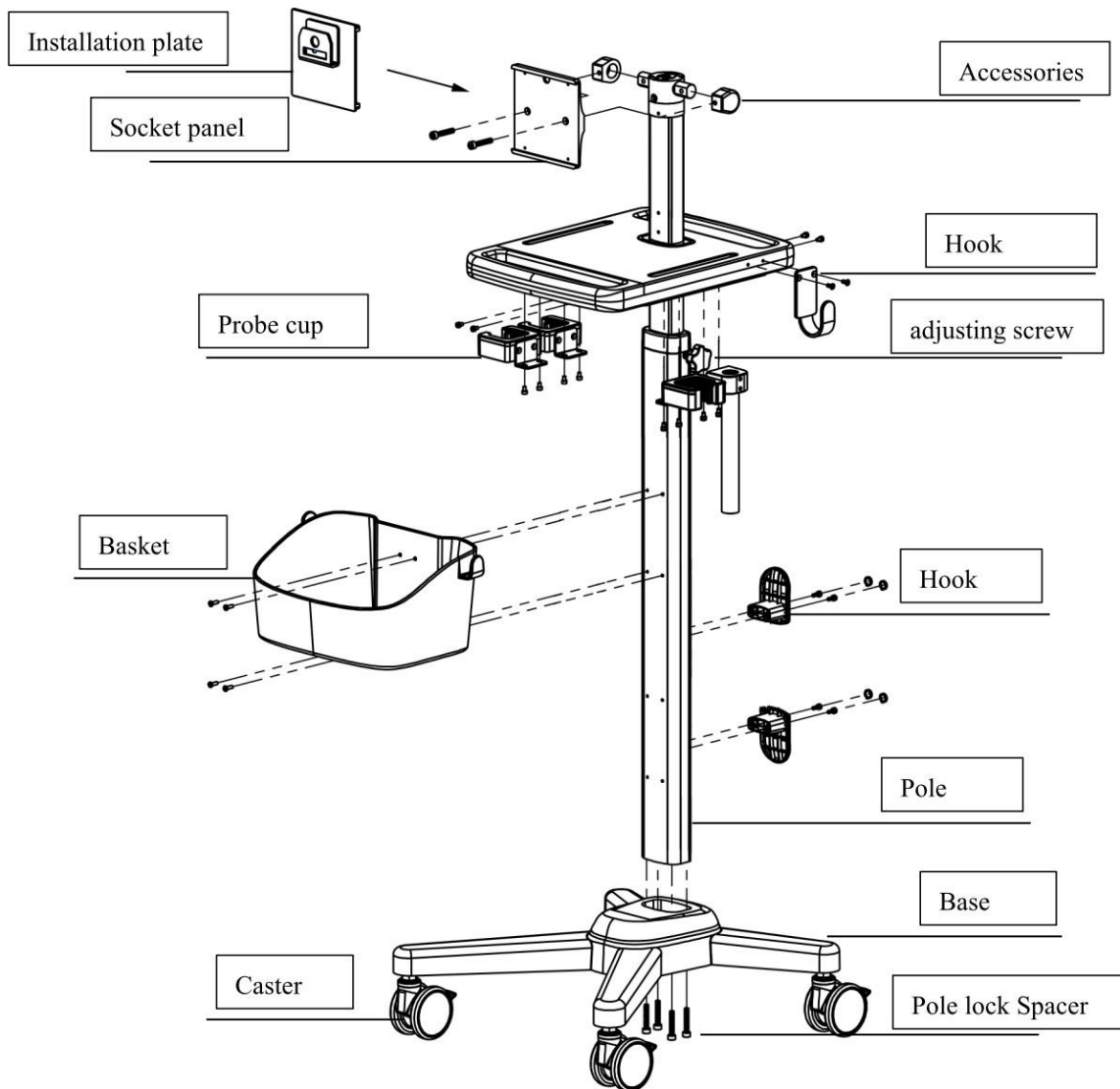9 TAW Trolley Stand Parts Schematic



10 Assemble the TAW Trolley Stand
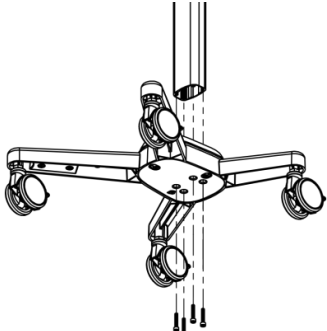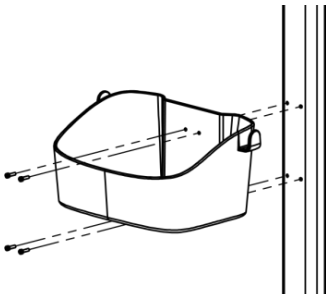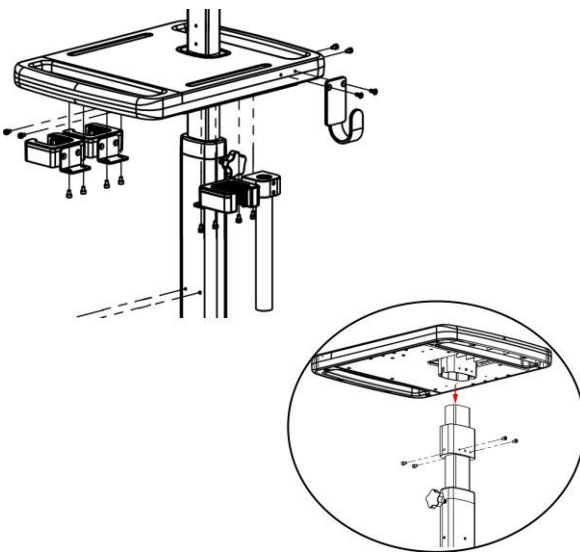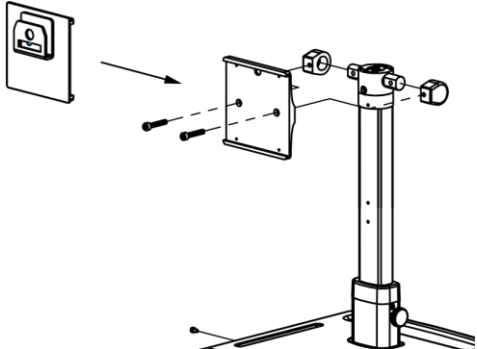
<p>1</p>	<p>Install the pole</p> <ul style="list-style-type: none"> - Insert TAW Trolley Stand Pole to the base - Use 4 PCS hex screws and spacer to lock the pole (M5*30mm) 	
<p>2</p>	<p>Install the basket</p> <ul style="list-style-type: none"> - Use 4 Allen socket screws to secure the basket to the pole mounting hole (M4*8mm) 	
<p>3</p>	<p>Install the desktop</p> <ul style="list-style-type: none"> - Use 6 machine-meter screws to mount the table top in the pole mounting holes (M5*8mm) <p>Install the hook</p> <ul style="list-style-type: none"> - Use 2 Self-tapping screws to mount the hook on the side of the table (M4*20mm) <p>Install the probe holder</p> <ul style="list-style-type: none"> - Use 8 hexagon socket screws to mount the probe holder and Round rod on both sides of the table (M4*10mm) 	
<p>4</p>	<p>Install the adapter</p> <ul style="list-style-type: none"> - Attach the adapter to the top of the pole with 4 Allen socket screws (M5*20mm) - Use 4 Round head hexagon socket screws to fix the rotating ball head on top of the pole (M5*35mm) as shown in the figure - Use 2 Allen socket screws to secure the fitting and chute plate to the rotating ball head (M5*20mm) 	

11 Maintenance and Adjustment.

Install the SourceMark VLMA Display onto the hooks of the mounting plate from top to bottom, and make sure it is firmly in place.

The TAW Trolley Stand requires very little maintenance during the warranty period, but we recommend regular inspection and maintenance at least every 12 months, and checking all screws and tightening them to their proper position if necessary.

This product has no predetermined expiration date. The service life depends on the frequency of use and maintenance conditions.

12 Cleaning and Disinfection

TAW Trolley Stand can be cleaned with common hospital cleaning agents (such as diluted bleach, ammonia, or alcohol solutions) and mild non-abrasive solutions.

The surface finish may be permanently damaged by aggressive chemicals and solvents such as acetone and trichloroethylene.

Do not use steel wool or other abrasive cloths for cleaning. Avoid any liquid from entering the interior of the Cart.

Note: UE does not make claims about disinfection effectiveness. Please consult with professionals in the infection control department of your hospital.