

BondEase® Topical Skin Adhesive

Quick Reference Guide

How to Use BondEase®

The BondEase® Topical Skin Adhesive (non-cyanoacrylate) is used to provide a secure, strong, flexible bacterial barrier for surgical incisions. Please review full Instructions for Use prior to use of BondEase® Topical Skin Adhesive.

1. Preparation for BondEase®


Clean and Dry: Thoroughly clean the wound site according to facility protocol. The wound must be completely dry, as moisture can accelerate polymerization and affect adhesion.

Positioning: Maintain the wound in a horizontal position whenever possible to prevent the liquid adhesive from flowing into unintended areas.


Approximation: Use deep dermal stitches if significant tension exists. Manually approximate the skin edges using gloved fingers or sterile forceps.

2. BondEase® Application

Activate: Hold the applicator tip facing upward.
(Fig 1.)

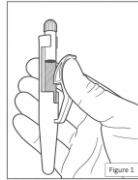

Squeeze: Press handle firmly to crush the internal glass ampoule and release.
(Fig 2.)

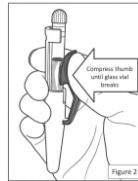

Start Flow: Invert the applicator. Gently squeeze the handle and hold until the blue liquid is visible passing through the filter within the tip.
(Fig. 3)



Apply: Using a gentle brushing motion, apply one continuous, uniform layer of adhesive over the approximated edges.
(Fig 4.)



Set Time: Maintain manual approximation for **two minutes**. The adhesive may remain tacky for up to four minutes; for easier film application wait until it is fully polymerized (no longer tacky) before applying a dressing.



Important Safety Precautions

Contraindications: Do not use on infected wounds, mucosal surfaces, or patients with polyurethane allergies.

Petroleum Jelly: Avoid use on the application site prior to bonding, as it will prevent the adhesive from sticking.

Post-Op Care: Patients may shower briefly but should not scrub, soak, or swim.

For assistance or to report an adverse event, contact **1-866-528-7001**.



BondEase®
Topical Skin Adhesive



Scan for link to IFU