



Single-Use Laryngoscope



Distributed by:
SourceMark, LLC
Franklin, TN 37067, USA
www.sourcemarkusa.com
(888) 786-4765

DIRECTIONS FOR USE

1. Fit blade onto handle until you hear a click and feel the blade snap into place to ensure the locking feature is engaged.
2. Lift blade until you hear a click and feel the blade snap into place to activate the light for operation
3. Unlock and disengage blade after use

Note: if needed, twist the handle end-cap counter- clockwise to remove the battery for disposal.

CAUTIONS

- Reuse of this device may cause injury and/or infection
- This device is intended for use on one patient during a single procedure and is not intended to be reprocessed (cleaned, disinfected, or sterilized) and used on another patient.
- Reprocessing of any single use device and any single use device with electrical contacts/terminals may cause device performance degradation or failure

INDICATIONS

Laryngoscopes are designed to provide illumination within the larynx during tracheal intubation.



Made in China
AB10-0010