


How to Hold Your Clipper:



- Base of clipper blade should be held against the skin
- For optimal results, clip against the grain of hair
- DO NOT flip clipper handle (SourceMark logo should always be facing upward)
- Always make sure skin is pulled tight in areas of sensitive or loose skin




How to Attach Your Clipper Blades:

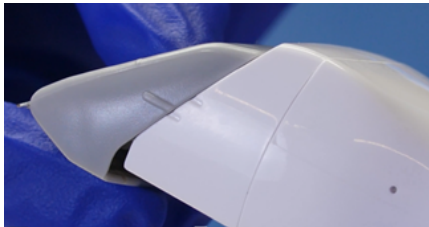


1

Match the line on the clipper blade up with the first line on the clipper handle.

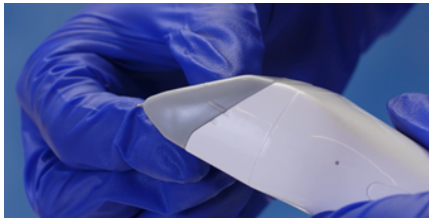

2

Push the clipper blade onto the clipper handle, the line on the blade should match up with the second line on the clipper handle.

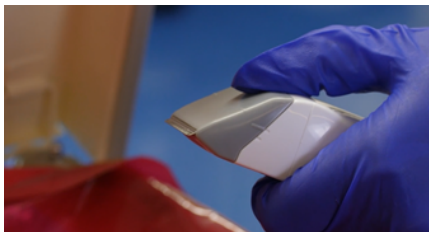

3

To dispose of the blade, place your thumb on the rigid indent of the blade and pop the blade off into the trash.



4

To clean the clipper of excess hair, run or dunk the clipper under water or wipe off with a non-corrosive disinfectant.