Français

Indication

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English

Indication

The Tourniquet Sleeve helps to protect the skin and soft tissues from wrinkling, pinching and shearing underneath Tourniquet Cuffs.

Instructions for use

The Sleeves are sized according to the VBM Cuff and the patient's limb circumference. The color coding helps to match the suitable VBM Cuff. The Sleeves are intended to fit snugly and extend beyond the edge of the Cuff to ensure that there is no direct contact between Cuff and skin. They apply a low pressure that is less than a snugly wrapped Cuff.

1. Use the VBM Measuring Tape to measure the patient's limb and determine the suitable Sleeve and Cuff size. The Measuring Tape is color coded and the size should match the color of the Sleeve rim and VBM Cuff strap.
2. Slide the Sleeve onto the limb ensuring that it is wrinkle-free.
3. Wrap the Cuff around the limb and over the Sleeve making sure that there is a safe distance between the cuff edges and the joints proximal and distal to the cuff.
4. Fold the distal portion of the Sleeve back over the Cuff.
5. After Cuff deflation immediately remove Cuff and Sleeve from the limb.

Attention

- Make sure to use the correct Sleeve size. A too narrow Sleeve can apply a too high pressure to the patient's limb. A too wide Sleeve can not be applied wrinkle-free and can cause injury to the patient after cuff inflation.
- After Cuff deflation immediately remove Cuff and Sleeve from the limb to allow venous return and to avoid congestion and pooling of blood in the operative field.
- For single use, do not re-use.
- Sterile packed, do not use if the package has visible damage.

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